



# B125P / B105P Patient Monitors

Powering your performance.



The B1x5P range of pre-configured patient monitors delivers premium clinical performance at an exceptional value. These accurate, reliable, and easy-to-use monitors enable simple and intuitive workflow with a choice of 10- or 12-inch touch screen displays across care areas.

## Advanced capabilities

B1x5P range of monitors can be deployed seamlessly across a variety of care settings:

- EK-Pro v14 ECG 4-lead simultaneous arrhythmia analysis
- DINAMAP™ SuperSTAT non-invasive blood pressure measurement
- GE TruSignal™ SpO<sub>2</sub> technology
- GE EtCO<sub>2</sub> sidestream measurement
- Connectivity to GE CARESCAPE™ networks

## Intuitive design. Uninterrupted workflow.

- 12 waveforms to view all required parameter waveforms simultaneously
- Bed to Bed communication and Automatic view on alarm (AVOA) to review remote patient monitoring data

- Roving functionality for seamless transition of the monitor from one bedside to another within the CARESCAPE Network
- InSite™ Remote Service platform for remote troubleshooting
- National Early Warning Score (NEWS) for timely intervention

## Tough for demanding duty. Secure for a cyber world.

- Follows FDA draft guidance for cyber security in medical devices
- An ECG filter delivers enhanced signal performance in noisy areas
- With High Capacity battery: >4 hrs<sup>1</sup>
- Tested with the EMC 4th Edition standard
- Water resistant with IP22 standards

## Technical specifications

### Display

|                           |  |
|---------------------------|--|
| Size                      | B125P: 12.1 in (diagonal)<br>B105P: 10.1 in (diagonal) |
| Resolution                | B125P / B105P: 1280x800 (WXGA)                         |
| Number of waveforms       | Up to 12   |
| Display layout and colors | User-configurable                                      |
| Controls                  | Capacitive touch screen and Trim Knob™                 |

### Parameters and modules

| Parameters                 | Modules <sup>2</sup>          |
|----------------------------|-------------------------------|
| ECG                        | Integrated hemodynamic module |
| Resp                       |                               |
| SpO <sub>2</sub>           |                               |
| NIBP                       |                               |
| Temp                       |                               |
| Sidestream CO <sub>2</sub> | E-miniC <sup>3</sup>          |

### ECG

|                     |   |
|---------------------|---|
| Leads available     | 3-lead configuration: I, II, III<br>5-lead configuration: I, II, III, aVR, aVL, aVF and V |
| Sweep speed         | 12.5, 25 or 50 mm/s   |
| Gain range          | 0.5x, 1x, 2x and 4x   |
| Heart rate accuracy | 20 to 300 bpm, ±5% or ±5 bpm, whichever is greater  |

### Bandwidth

|                     |   |
|---------------------|---|
| ECG filter          | Monitor: 0.5 to 40 Hz<br>ST: 0.05 to 40 Hz<br>Diagnostic: 0.05 to 145 Hz<br>Moderate: 0.5-20 Hz |
| Pacemaker detection | Voltage range: 2 to 700 mV<br>Pulse width: 0.5 to 2 ms  |

### Arrhythmia Alarms

|                    |   |
|--------------------|---|
| Lethal Alarms      | Asystole, V Fib/ V Tach, V Tach   |
| HR Alarms          | Brady, Tachy  |
| Ventricular Alarms | VT>2, R on T, V Brady, Couplet, Bigeminy, Accelerated Ventricular, Trigeminy, Multifocal PVCs |
| Atrial Alarms      | A Fib, Missing beat, Pause, Irregular, SV Tachy   |
| PVC Alarm          | Frequent PVCs, Frequent SVCs  |

### ST segment analysis

|                    |   |
|--------------------|---|
| Numeric range      | -9 to +9 mm (-0.9 to +0.9 mV)   |
| Accuracy           | ±0.2 mm or ±10%, whichever is greater, within the measurement range of -8 to 8 mm |
| Numeric resolution | 0.1 mm (0.01 mV)  |

### Impedance respiration

|            |  |
|------------|--|
| Range      | Adult/pediatric: 4 to 120 breaths/min<br>Neonate: 4 to 180 breaths/min |
| Accuracy   | ±5% or ±5 breaths/min, whichever is greater                            |
| Gain range | 0.1 to 5 cm/Ohm  |

### SpO<sub>2</sub>

#### TruSignal SpO<sub>2</sub>

##### Measurement range

|                      |               |
|----------------------|---------------|
| Pulse oximetry       | 1 to 100%     |
| Pulse rate           | 30 to 250 bpm |
| PI (Perfusion Index) | 0 to 32       |

##### Measurement accuracy

|            |  |
|------------|--|
| Saturation | Without motion-adult/pediatric<br>Finger sensor: 70 to 100% ±2%<br>Without motion-neonate:<br>70 to 100% ±3%<br>With motion-adult/pediatric/<br>neonate: 70 to 100% ±3%<br>Low perfusion-adult/pediatric:<br>70 to 100% ±3% (<70% unspecified) |
| Pulse Rate | Without motion: ±2 bpm<br>(Adult/Pediatric/Neonatal)   |

### NIBP

|                       |  |
|-----------------------|--|
| Measurement technique | Oscillometric with step deflation                              |
| Measurement Modes     | Manual, Automatic (with customseries cycle time), and STAT     |
| Automatic Cycle Times | Custom, 1, 2, 3, 4, 5, 10, 15, 20, 30 min, 1 h, 1.5 h, and 2 h |

### NIBP Measurement ranges

|           |  |
|-----------|--|
| Systolic  | Adult/Pediatric: 30 to 290 mmHg<br>Neonate: 30 to 140 mmHg |
| MAP       | Adult/Pediatric: 20 to 260 mmHg<br>Neonate: 20 to 125 mmHg |
| Diastolic | Adult/Pediatric: 10 to 220 mmHg<br>Neonate: 10 to 110 mmHg |

### Clinical Accuracy

|                    |   |
|--------------------|---|
| Mean Difference    | ±5 mmHg                                 |
| Standard Deviation | ≤ 8 mmHg                                |
| Reporting Standard | ANSI/AAMI ISO81060-2 and IEC 80601-2-30 |

<sup>2</sup> Refer to B105M/B125M/B155M User's Manual for more information.

<sup>3</sup> CO<sub>2</sub> measurement through E-miniC Module is intended for use with patients weighing over 5kg (11 lb) only.

## Safety features

|                                    |  |
|------------------------------------|--|
| Default initial inflation pressure | Adult/Pediatric: 135 ±15 mmHg<br>Neonate: 100 ±15 mmHg             |
| Maximum determination time         | Adult/Pediatric: 2 min<br>Neonate: 85 s                            |
| Over pressure monitor              | Adult/Pediatric: 300 ±6 to 330 mmHg<br>Neonate: 150 ±3 to 165 mmHg |

## Pulse Rate from NIBP

|                   |   |
|-------------------|---|
| Measurement Range | 30 bpm to 250 bpm                       |
| Accuracy          | ±5% or ±5 bpm<br>(whichever is greater) |

## Temperature

|                   |         |
|-------------------|---------|
| Numerical display | T1, T2, |
|-------------------|---------|

## From integrated hemodynamic measurement (T1, T2)

|                      |   |
|----------------------|---|
| Measurement range    | 10 to 45°C (50 to 113°F)  |
| Measurement accuracy | ±0.1°C without probe<br>±0.2 °C with probe from 25 to 45 °C<br>±0.3 °C with probe from 10 to 25 °C<br>(not include 25 °C) |
| Display resolution   | 0.1°C   |

## Network architecture

|              |                   |
|--------------|-------------------|
| Physical N/W | 1000BaseT network |
|--------------|-------------------|

## Networking services

|                   |  |
|-------------------|--|
| Outbound HL7®     | Direct Connectivity to EMR or 3rd party systems for numeric trend                    |
| CARESCAPE (Unity) | Connectivity to CIS / HIS through CARESCAPE Gateway<br>Other Networking applications |
| Remote Service    | Remote Diagnosis of device via InSite™ RSvP server                                   |

## CARESCAPE (Unity) networking applications

### Bed to Bed window\*

|                       |  |
|-----------------------|--|
| Data displayed        | Six parameters' waveforms and numeric values, one remote alarm, and remote bed information |
| Remote beds Monitored | Monitor alarms for up to 40 beds<br>View one bed from up to 1023 beds                      |

### AVOA (Auto View of Remote beds in alarm)\*

|                                  |   |
|----------------------------------|---|
| Remote alarm message information | Unit and bed name, alarm message, more than 1 beds alarming |
| Configurable alarm notification  | Message, Auto View, Auto View Always                        |

## Roving

|               |   |
|---------------|---|
| Functionality | Roving between units and beds;<br>Adding new units and beds;<br>Selecting the printer |
|---------------|---|

## I/O Peripherals

### Standard Connectors

|               |   |
|---------------|---|
| Ethernet port | Supports HL7 and CARESCAPE Unity N/W  |
| USB 2.0 Port  | Download service logs<br>Import / Export settings<br>Export numerical trends<br>Installing software, firmware and e-manuals |
| HDMI Port     | Supports secondary clone display<br>1280 x 800 pixels   |

### Non-standard Connectors

|                    |  |
|--------------------|--|
| Recorder Connector | Standalone thermal printer B1X5-REC Recorder |
|--------------------|--|

## Network and data security

|                   |  |
|-------------------|--|
| LAN Connection    | Supports IEEE 802.1X port-based Network Access Control (NAC)   |
| USB file exchange | All USB functions are password protected<br>Encrypted export of numerical trends, user settings, and service logs to USB |

## Mounting

|                            |  |
|----------------------------|--|
| GCX compatible             |  |
| Integrated carrying handle |  |

## Local thermal printer

|                         |  |
|-------------------------|--|
| Method                  | Thermal dot array  |
| Horizontal resolutions  | 24 dots/mm (600 dpi)   |
| Vertical resolution     | 8 dots/mm (200 dpi)  |
| Waveforms               | Selectable 1, 2, or 3 waveforms  |
| Numerics trend printout | HR, Pleth, NIBP, T1, T2, EtCO <sub>2</sub> , and RR                      |
| Paper width             | 50 mm, printing width 48 mm  |
| Paper speed             | 5, 10, 12.5 and 25mm/s, user configurable                                |
| Remote printer          | Supports both Laser and thermal printer (with CARESCAPE Central Station) |

## Module Rack (Optional)

|                          |  |
|--------------------------|--|
| Slot for a single module |  |
|--------------------------|--|

\* Compatible only with B155M / B125(M/P) / B105(M/P) VSP3.0 patient monitors

## Performance specifications

---

### Alarms

|                           |   |
|---------------------------|---|
| Priority                  | Adjustable priority: High, Medium, Low and Information<br>Local and remote control from central station |
| Alarm breakthrough        | Asystole, V Fib/V Tach, V Tach, Brady   |
| Alarm configurability     | Define VTach rate range and duration criteria for a sustainable VTach alarm                             |
| Notification              | Audible and visual  |
| Alarm tone                | IEC, General, ISO, ISO2   |
| Setting                   | Default and individual  |
| Visual alarm notification | Red, yellow, cyan<br>Audio silence message<br>General alarm message                                     |
| Alarm limit adjustment    | Local and remote control from central station   |
| Audio pause               | 2 min   |
| Alarm auto printing       | Up to 5 alarms  |

### Trends

|              |  |
|--------------|--|
| Graphical    | All parameters, selectable time scales from 20 min to 168h (7 days)  |
| Numerical    | All parameters, with 168 hours (7 days) of trend data sampling according to time setting or after NIBP determination                                   |
| Snapshot     | Up to 200 snapshots Manual or alarm triggered<br>Event snapshots with waveform (on CARESCAPE Central Station)  |
| OxyCRG trend | Neonate mode only<br>Real time or snapshot view<br>Stores up to 70 OxyCRG snapshots<br>Snapshot duration 6 min before and 2 min after the OxyCRG event |
| Trend cursor | In graphical trend   |

### Full disclosure

#### Tab/page: all ECG, Hemo

|  |  |
|--|--|
| All ECG view                             | ECG I, II, III, aVL, aVR, aVF, and V waveforms |
| Hemo view                                | ECG II, SpO <sub>2</sub> and Resp waveforms    |
| Parameters supported                     | ECG, SpO <sub>2</sub> and RESP                 |
| Configurable waveform review sweep speed |  |
| Storage                                  | 72 hours with all waveform data                |
| Integrated link with alarm history       |  |
| Full Disclosure review on specific alarm |  |
| Full Disclosure review on specific time  |  |

## EWS (Early Warning Score)

|   |   |
|---|---|
| Protocol  | National Early Warning Score (NEWS) 2   |
| Parameters  | Pulse HR/PR, Systolic Blood Pressure, LOC (level of consciousness), TEMP, SpO <sub>2</sub> , Resp Rate, and Air or Oxygen |
| History with detailed parameters values and sub-scores                              |   |
| Total EWS score on the main screen with color coding and time stamps                |   |
| Clinical response and individual parameter scores with colors on a dedicated window |   |
| Review EWS Clinical Risk and EWS Guidance   |   |

## Environmental specifications

---

### Operating conditions

|                      |                                   |
|----------------------|-----------------------------------|
| Temperature          | 5 to 40°C (41 to 104°F)           |
| Relative humidity    | 15 to 90% non-condensing          |
| Atmospheric pressure | 700 to 1060 hPa (525 to 795 mmHg) |

### Storage and transport conditions

|                      |                                   |
|----------------------|-----------------------------------|
| Temperature          | -20 to 60°C (-4 to 140°F)         |
| Relative humidity    | 10 to 90% non-condensing          |
| Atmospheric pressure | 700 to 1060 hPa (525 to 795 mmHg) |

## Power specifications

---

|                   |  |
|-------------------|--|
| AC input          | 100 to 240V ±10%, 50/60 Hz   |
| Power consumption | Monitor ≤150 VA  |
| Protection        | Class I  |
| Battery           | 1 Lithium Ion - option from basic & high capacity  |
| Charging time     | < 4 h to 90% capacity  |
| Run time          | Battery backup time*<br>High capacity: >4.0 hrs for B125P<br>>4.5 hrs for B105P<br>Basic battery: >2.0 hrs for B125P<br>>2.5 hrs for B105P |

## Physical specifications

---

### Monitor

|                                       |  |
|---------------------------------------|--|
| Dimensions (H x W x D)                | B125P: 280 x 312 x 175 mm<br>B105P: 275 x 265 x 175 mm |
| Weight (with battery and w/o modules) | B125(P): ≤ 4.2 kg<br>B105(P): ≤ 3.8kg                  |
| Ingress protection                    | IP22   |

\* With typical configuration: ECG, NIBP cycle time 15 min, SpO<sub>2</sub>, display brightness 70%



## Certifications

---

IEC 60601-1 passed

CE marking according to EU Medical Device Regulation (EU) 2017/745

UL mark

CB certificate

## System

---

|                  |   |
|------------------|---|
| Operation system | Linux®  |
| Cooling system   | Natural convection, no fan inside for cooling |

Product may not be available in all countries and regions. Full product technical specification is available upon request. Contact a GE Healthcare Representative for more information. Please visit [www.gehealthcare.com/promotional-locations](http://www.gehealthcare.com/promotional-locations)

Data subject to change.

© 2020 General Electric Company.

GE, the GE Monogram, CARESCAPE, DINAMAP, Entropy, Trim Knob and TruSignal are trademarks of General Electric Company.

Masimo and SET are trademarks of Masimo Corporation. Nellcor and OxiMax are trademarks of a Medtronic company. HL7 is a registered trademark of Health Level Seven (HL7), Inc. Linux is the registered trademark of Linus Torvalds in the U.S. and other countries. All other third-party trademarks are the property of their respective owners.

Reproduction in any form is forbidden without prior written permission from GE. Nothing in this material should be used to diagnose or treat any disease or condition. Readers must consult a healthcare professional.

B105P, B125P

DOC2378611 2020-09-17

*B105P / B125P patient monitor software version VSP3.0 are not available in all markets and are not 510k cleared.*